(DANISH)
BLU-MOUSSE SUPERFAST INSTRUKTIONER

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(ENGLISH)
BLU-MOUSSE SUPERFAST INSTRUCTIONS

BLU-MOUSSE SUPERFAST: a radiopaque, thixotropic (resists slumping), addition-reaction silicone (polyvinyl siloxane) designed for use as a bite registration material. No resistance to bite closure. Available in automix cartridges only. Sets intra-orally in thirty (30) seconds. Compatible with other polyvinyls. Contains no hazardous ingredients. May be disinfected in 2% glutaraldehyde solution.

Working Time @ 23°C (73°F)	30 seconds
----------------------------	------------

Minimum Intra-oral Setting Time	30 seconds
Final Durometer	85 (Shore A)

INSTRUCTIONS

- Insert cartridge into impression gun, twist off sealing cap, express approximately 5mm (1/4”) of material to insure equal flow. If apertures are clogged, clear blockage with an explorer before proceeding.
- Securely attach mixing tip (1/4 turn) and squeeze trigger with moderate, even pressure to extrude mix directly onto occlusals of teeth. Guide patient into centric occlusion to register bite.
- After extruding material, leave mixing tip attached as sealing until next use.

IMPORTANT! Like other polyvinyls, Blu-Mousse Superfast emits hydrogen after setting. For best results, delay pouring model for three (3) hours.

STORAGE AND USAGE

- Store between 18°C and 24°C (64°F-75°F). Exposure to excess heat will shorten normal shelf-life (12 months).
- For best results, use at normal room temperatures. For extended work/set times, use Blu-Mousse Classic (2-minute intraoral set; Stock No. S438).
- Contamination by latex gloves can interfere with the setting reaction of polyvinyls. Polyvinyls are not compatible with polyether and polysulfide materials. Acrylic and methacrylate residues might inhibit setting of polyvinyls.
- Hard-setting Blu-Mousse Superfast may lock into severe undercuts. Block them out. Advise laboratory to trim registrations prior to mounting models so that cusp tips fully seat into record.

ORDERING INFORMATION

Stock No. S436 - Blu-Mousse Classic (2 minute set)
Tubes, contains 1 tube base, 80ml, and 1 tube catalyst, 80ml.
Stock No. S438 - Blu-Mousse Classic (2 minute set)
Automix Cartridges, contains 2 - 50ml cartridges plus mixing tips.

Stock No. S457 - Blu-Mousse Super-Fast (2 minute set)
Automix Cartridges, contains 2 - 50ml cartridges plus mixing tips.

WARRANTY

Parkell will replace, free of charge, any material found to be defective for a period of 1 year from date of manufacture if handled and stored properly. Parkell does not accept liability for any loss or damage, direct or consequential, from the use of or the inability to use the product described herein. The user shall determine the suitability of the product’s intended use, and the user assumes all risk and liability in connection therewith. There is no implied warranty connected with the sale of this merchandise.

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