



Rx Only
INSTRUCTIONS FOR USE

INTENDED USE/INDICATIONS:

For use in the extra-oral adjustment of impressions, soft denture liners and dental elastomeric materials.

CONTRAINDICATIONS:

Hedgehog Burs are NOT FOR USE ON LIVE PATIENTS, OR IN THE MOUTH. They are contraindicated for use on hard dental acrylics or dental stone materials, which will render the bur unusable by clogging the abrasive surface.

PARKELL'S HEDGEHOG BUR LINE INCLUDES:

S497 – Flat-End Cylinder

S498 – Pear

S499 – Ball

S500 – Round-End Cylinder

S501 – Long Taper

S502 – Taper

S503 – Saw-Wheel

WARNINGS:

- Hedgehog Burs should only be used by qualified, licensed dental professionals.
- Hedgehog Burs are sharp cutting instruments, and can be dangerous when rotating or stationary. Exercise caution and wear facemask and eye protection during use.

- The shank of the Hedgehog bur must be fully seated in the straight-shank chuck of a slow-speed handpiece to prevent bending during use. Do not use at speeds above 35,000 rpm.
- Disinfect the appliance or impression before adjustment, using appropriate CDC guidelines.
- Remove rubber gloves before using the Hedgehog Bur. The rotating spikes of the abrasive surface can catch the rubber glove and pull your finger into the bur.
- Discard Hedgehog Burs in hazardous waste receptacle when they are no longer cutting effectively.

CLEANING AND STERILIZATION GUIDELINES:

- Hedgehog Burs are NON-STERILE when received. They should be sterilized BEFORE first use, and between uses on different patients, appliances or impressions.
- Before autoclaving Hedgehog Burs, remove debris by scrubbing with a brush or cleaning in an ultrasonic cleaner with warm, soapy water, rinsing with water and blotting dry with a paper towel.
- Hedgehog Burs should be bagged in autoclave indicator bags and

Continued...

sterilized in any conventional steam autoclave (132-135° C for 15 min for gravity displacement or 4 minutes for pre-vacuum cycle, or using the manufacturer's recommended procedure, if different). Minimum cool down interval is 15 min.

- For further information, consult the Centers for Disease Control website at CDC.gov, referencing the most recent version of the "Guidelines for Infection Control in Dental Health-Care Settings".

WARRANTY:

Hedgehog Burs are consumable items, and no warranty is expressed or implied. They are non-refundable and non-returnable if the package has been opened. Parkell does not accept liability for any loss or damage, direct, consequential or otherwise, arising out of the use of or the inability to use the product herein described. Before using, the user shall determine the suitability of the product for its intended use and the user assumes all risk and liability whatsoever in connection herewith.

Parkell's Quality System is certified to ISO 13485.



European Authorized Representative (Not a dealer/distributor):
Directa AB - P.O. Box 723, Finvids väg 8
SE-194 27 Upplands Väsby - Sweden

parkell[®]

300 Executive Dr., Edgewood, NY 11717 USA

Toll-Free: 1-800-243-7446 • **Phone:** (631) 249-1134

Fax: (631) 249-1242 • **E-Mail:** info@parkell.com

Web: www.parkell.com

Emergency Phone (24 hrs.):

INFOTRAC: (North America) 1-800-535-5053

(International) 1-352-323-3500

