EN

Rx Only—product is intended for use by a licensed dental professional.

Refer to TurboSensor® (REF D560) Instructions for Use for sterilization method and 'Explanation of Symbols Used' glossary.

Autoclavable Handpiece Sheaths (Qty: 2)

This scaler is provided with two autoclavable handpiece sheaths (REF D576); one attached to the handpiece, and one spare in this bag. Prior to use, both sheaths should be sterilized following the "Cleaning and Infection Control" recommendations in the Instructions for Use.

- To remove the sheath from the handpiece, hold the handpiece at the base with one hand and turn the sheath counter-clockwise with the other, taking care not to twist the handpiece hose. The sheath will unscrew and slip off.
- To replace the sheath, gently thread it onto the handpiece clockwise, until there is no gap between the two. Take care not to damage or dislodge the O-ring. Do not overtighten.

Water Filter Disks (Qty: 2)

This scaler is provided with two water filter disks (REF D419); one attached to the rear water line hose, and one spare in this bag. The Luer-Lock water filter disk should be replaced when it becomes dark or clogged with debris so that water flow is not blocked (1-3 months). Replacement filters are available from Parkell.

To remove:

- Close any water valves, and bleed all water lines of residual water pressure.
- Disconnect the scaler water supply hose from the water supply at the quick-connect.
- Remove the old filter disk by holding it in one hand and twisting it off the hose connectors on either side in opposite directions.
- Install the new filter disk in the same position as the old one, and tighten the connectors in reverse.
- Reconnect the scaler water supply hose to the water supply via the quick-connect and check for leaks.

If the filter clogs and you do not have a replacement, the connectors may be locked together TEMPORARILY without the filter for emergency operation.

Replacement Orange O-Rings (Qty: 2)

- Use if water leaks at base of handpiece, by replacing O-ring (REF D588) at nose of handpiece.
- Moisten O-ring with water to allow easy replacement and smooth, tight connection.

Self-Adhesive Handpiece Holder (Qty: 2)

- Determine desired location for handpiece to hang.

- Peel off paper from self-adhesive back and apply holder (REF D323) to surface of desired location. Apply pressure for 30 seconds.

- Wait 24 hours before applying weight to the holder to allow maximum bond strength.

Questions?

If you have any questions or problems, please call Parkell Technical Support at (800) 243-7446, M-F from 8:30 AM to 5:00 PM EST.

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**Parkell, Inc.,**
**UKRP: Topdental (Products) Ltd.,**