




# Standard\* Ultrasonic Insert Wear Guide

Fig.1

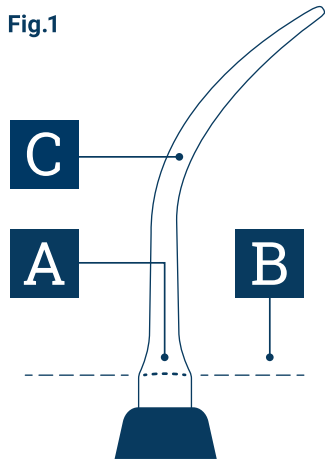

## Check Your Inserts Regularly!

Worn tips can greatly decrease your scaling efficiency, and can necessitate the use of excessive pressure during scaling which can result in discomfort for both you and for your patient.

**\* PLEASE NOTE:** For DuraTip® inserts, this wear guide is not applicable and should not be used. Instead, DuraTip Performance Guides should be used exclusively with DuraTip inserts.


## How to Use This Wear Guide

1. Locate the diagram of the insert to be measured (use the SKU# which is printed on the insert's metal stack)
2. Align point (A) of the insert to line (B) on the Wear Guide (see Fig. 1)
3. Holding insert tip flat against the wear guide, align the curvature of tip (C) to the silhouette graphic
4. Evaluate tip condition, comparing length to blue/magenta lines.



### Peak Efficiency

Tip extends past blue line

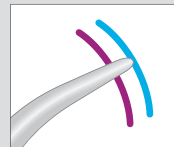

### Reorder

When tip touches blue line (indicates 25% efficiency loss)



### Discard

When tip touches magenta line (indicates 50% efficiency loss)

This guide is intended to assist you in measuring your Parkell® ultrasonic inserts for wear. Measurements are subject to variations in manufacturing and user handling of both the insert and the guide, and therefore results may vary. Responsibility for determining when to replace a tip resides with the user. This guide should not be used for inserts that have been subjected to improper use or whose tips have been bent or reshaped as it will not provide reliable information.

A01223-A1123



# Standard\* Ultrasonic Insert Wear Guide



Parkell, Inc., 300 Executive Dr., Edgewood, NY 11717  
+1 (800) 243-7446 • info@parkell.com • Parkell.com

**DTI30**

30k Burnett Power-Tip™

**DPI30**

30k Perio

**DCG30UNI**

30K Comfort Grip / Universal

**DCG30TRI**

30K Comfort Grip / Triple Bend

**DUI30**

30k Universal

**DTI25**

25k Burnett Power-Tip™

**DCG30SLIM**

30K Comfort Grip / Universal Slim

**DPI25**

25k Perio

**DSG30T**

30k Soft-Grip Burnett Power-Tip™

**DUI25**

25k Universal

**DSG30P**

30k Soft-Grip Perio

**DSG30U**

30k Soft-Grip Universal

**\*PLEASE NOTE:** For DuraTip® inserts, this wear guide is not applicable and should not be used. Instead, DuraTip Performance Guides should be used exclusively with DuraTip inserts.