

# IMPORTANT USER INFORMATION

The Parkell TurboPIEZO™ handpiece IS NOT HEAT STERILIZABLE. It will be damaged if sterilized in a steam autoclave, dry heat unit or other heating device. Heat sterilization of the handpiece will void the unit's warranty.



## DISINFECTION INSTRUCTIONS

The TurboPIEZO handpiece should be thoroughly disinfected after each patient, using the following procedure:

1. Remove scaling tip using the Piezo Tip Wrench. Sterilize tip and wrench using the protocol for your particular autoclave. See the TurboPIEZO instructions for details.
2. Wet a paper towel with an EPA-approved high-level surface disinfectant and wipe down the handpiece for the disinfectant manufacturer's suggested time.

**NOTE: Do not spray disinfectant directly on the handpiece, as it may seep into the mechanism and cause corrosion damage.**

3. Wipe the disinfected handpiece with a water-moistened paper towel to remove the disinfectant.
4. Wipe the damp handpiece with a dry paper towel and store the handpiece in a clean location.


During treatment, Parkell recommends that you use the handpiece with an appropriate plastic barrier protection. When using barriers such as the Aurora™ Barrier Sleeves (REF S130), the scaling tip should protrude through a puncture

made in the closed end of the sleeve. The only part of the handpiece that is exposed to oral fluids is the scaling tip, which should be autoclaved after each patient.

## TIP TIGHTENING INSTRUCTIONS

The Piezo Tip Wrench contains a friction clutch which protects against over-tightening the tip. Prior to using the Piezo unit, hand-thread a tip onto the end of the handpiece until finger-tight—be careful not to cross the threads. Insert the working end of the tip into the square hole on the scalloped side of the tip wrench. Turn the wrench clockwise until the clutch slips and the tip stops turning. Remove the wrench and the unit is ready for use. Tip removal is accomplished by turning the wrench in a counterclockwise direction.

**If you have any questions, please e-mail our Technical Support Service at: [techsupport@parkell.com](mailto:techsupport@parkell.com)**

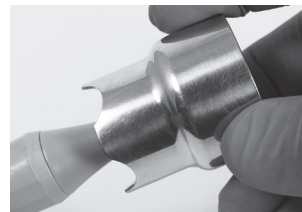

**parkell**®

300 Executive Dr., Edgewood, NY 11717, USA • (800) 243-7446 • [Parkell.com](http://Parkell.com)

A00600revB1019