

# Brush&Bond MAX

4-META Dentin/Enamel Composite Bonding System



DENTAL  
PRODUCTS  
REPORT



## Touch Application™ –No Scrubbing!

### The Story Behind Touch Application™

When over 55% of doctors surveyed, who currently use a bonding agent that requires a scrub application technique, told us they did not feel confident that they are covering the entire prep surface, and 30% of those same doctors felt incomplete scrubbing may be a contributing factor when their patients experience post-op sensitivity, **we knew we had a better way.**

Brush&Bond® MAX's Touch Application™ technique eliminates the need to scrub the prep, providing consistent, even, and complete coverage of the entire prep surface. **Simply dip the chemically treated Brush&Bond® Activator Brush into 1-3 drops of liquid and touch it to the prep surface until the entire prep is wet.** This easy application process helps ensure MAXimum adhesion between the tooth and the restorative, while virtually eliminating post-op sensitivity.



**Touch Application™**



**Eliminates Post-Op Sensitivity**




**Higher Bond Strength**

### Benefits

- ✓ **Touch Application™**— removes the doubt often associated with scrub-application products by ensuring consistent, even, and complete coverage of the entire prep surface
- ✓ **Higher bond strengths** to enamel and dentin than the leading competitors
- ✓ **MAXimum** sealing of, and penetration into, the dentinal tubules to alleviate concerns of post-op sensitivity
- ✓ Enhances the bond of most self-adhesive resin cements when used as a prep primer prior to the cementation of crowns on short, tapered, low-retention preps (especially beneficial for 2nd molars)
- ✓ Extensive penetration into the tubules promotes the formation of a robust resin-tag network that ensures MAXimum adhesion between the tooth and the restorative
- ✓ Optimized for adhesion to the surfaces dentists use their bonding agent on the most: enamel and dentin

### Features

- ✓ Single-bottle system that combines etchant, adhesive, and desensitizer all in one<sup>1</sup>
- ✓ Works with all etching techniques
- ✓ Compatible with all composites
- ✓ Low film thickness
- ✓ Single coat application
- ✓ Primer for sandblasted metal
- ✓ Works with all currently available curing lights

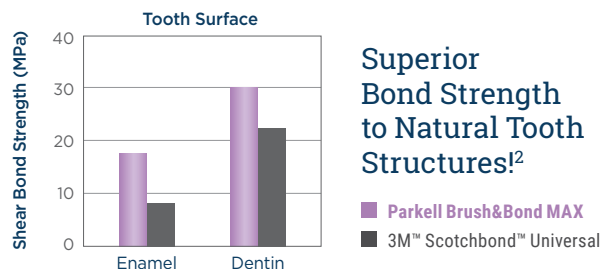

**Superior Bond Strength to Natural Tooth Structures!<sup>2</sup>**

SKU#	DESCRIPTION
S220	Brush&Bond Max Kit; (1) 3ml bottle of Brush&Bond Max Liquid, (100) Standard Activator Brushes, (1) Mixing Well
S222	Brush&Bond Max Liquid (3ml bottle)

ACCESSORIES & REFILLS	
S286	Brush&Bond Standard Activator Brushes (100 brushes)
S287	Brush&Bond Mini/Endo Activator Brushes (100 brushes)
S373	Plastic Mixing Well (2-pack)

1. Must be used with Brush&Bond Standard Activator Brushes.  
2. Data on file. Trademarks are property of their respective companies.

For more information, visit [Parkell.com](http://Parkell.com) or contact your authorized Parkell distributor.

