

Predicta® core

BIOACTIVE


Dual-Cure Build-Up Material



Dual-Cure Composite for Core Build-Ups and Post Cementations


Optimized for placement in teeth that require core build-ups, with or without posts, prior to the prepping, impressioning, fabrication, and cementation of a crown.

- ✓ Predicta® Bioactive Core is bioactive and releases calcium and phosphate ions, and releases and recharges fluoride ions to stimulate mineral apatite formation and remineralization at the material-tooth interface
- ✓ Dual-cure for unlimited depth of cure
- ✓ Zirconia added to improve the prepping experience
- ✓ Indicated for core build-ups and as a post cement
- ✓ Strong and stable support for crowns made of zirconia, porcelain, PFM and metal
- ✓ Multiple viscosities

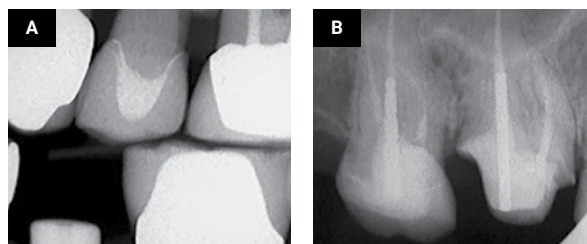



A: Tooth #13 — A void-free Predicta Core provides strength and support for an e.max crown. **B:** Tooth #3 — Post cementation and core build-up accomplished with Predicta Bioactive Core.

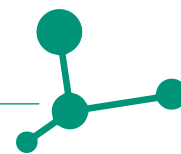



(Michael Karter, DDS)



What is Bioactivity?

Known in the field to equate to stronger bonds between the restoration and tooth, penetration and filling of micro-gaps, reduction in sensitivity, guard against secondary caries, and the sealing of margins against microleakage.

	
<p>Stackable (High Viscosity)</p> <p>Ideally suited to minimize crown prepping time, when placing larger core build-ups where stackability results in a more centralized core, and less excess in areas that would require removal during prepping.</p>	<p>Flowable (Low Viscosity)</p> <p>Recommended for cases where post cementation and core build-ups are done together. The lower viscosity promotes better canal adaptation and less hydrostatic resistance during seating of the post into the canal.</p>
<p>Both Stackable and Flowable Predicta Core versions are appropriate for, and can be used in all core build-up and post cementation situations. These guidelines should be considered an aid when choosing which build-up material more closely matches your needs and preferences.</p>	

SKU#	DESCRIPTION	Predicta Bioactive Core Build-Up Kits include: <ul style="list-style-type: none"> • (1) 5 ml hand-held syringe • (10) 17-gauge mixing tips • (10) 19-gauge mixing tips
	S600 Flowable / White Shade	
	S601 Flowable / Tooth Shade	
	S605 Stackable / White Shade	
	S606 Stackable / Tooth Shade	

ACCESSORIES & REFILLS

S620	5 ml Dispensing Gun
S621	17-Gauge Dispensing Tip Refill (30 pcs)
S622	19-Gauge Dispensing Tip Refill (30 pcs)



For more information, visit Parkell.com or contact your authorized Parkell distributor.

parkell®