

# Predicta<sup>®</sup> desensitizer

## BIOACTIVE

Desensitizer Gel



*"It doesn't get better than this!"*

— Helen Lester, DDS

After realizing Predicta<sup>®</sup> Bioactive Desensitizer truly changed her patient's life when she received a personalized thank you note, Dr. Lester now offers Predicta Bioactive Desensitizer as a regular treatment for patients with dentin hypersensitivity. **"To reduce or eliminate patient discomfort is a significant boost for my patients. This product delivers!"**



## The Only FDA-Cleared Bioactive Desensitizer

Predicta Bioactive Desensitizer is changing the way clinicians treat dentin hypersensitivity by using the elements that are naturally found in the tooth, (calcium and phosphate) and restoring them to the tooth in the form of hydroxyapatite plugs that continue to form in, and cause occlusion of, dentinal tubules.

As calcium and phosphate ions continue to release, mineralization occurs atop the dentinal tubules to seal the tubules and mineral apatite plugs continue to form and extend down into the dentinal tubules to create an expanded zone of protection.


Predicta Bioactive Desensitizer is also considered highly biocompatible as there are no methacrylates or soft-tissue irritating chemicals used during production.

### Treats Dentin Hypersensitivity Due To:

- ✓ Exposed roots after perio surgery
- ✓ Bleaching procedures
- ✓ Scaling or root planing
- ✓ Hypersensitivity due to gingival recession

### Easy-to-Use, Easy-to-Apply!

This syringe dispensed gel requires no mixing. Simply remove any calculus or plaque from the treatment site, blot the area dry with a cotton roll or gauze and you are good-to-go.

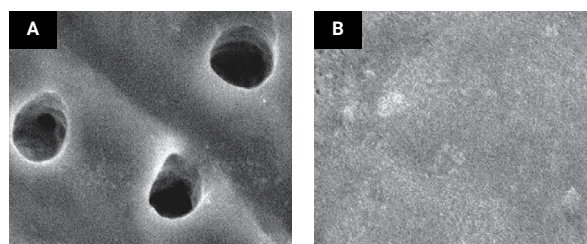

Comparison of untreated dentinal tubules (A), with tubules that have been covered with a dense layer of Predicta Bioactive Desensitizer (B). (University of Washington School of Dentistry)

SKU#	DESCRIPTION	
S626	<b>Single Syringe Kit;</b> (1) 3 ml syringe, (10) applicator tips, (15) applicator brushes, (1) mixing well	<b>Predicta Bioactive Desensitizer Kits</b>
S625	<b>Three Syringe Kit;</b> (3) 3 ml syringes, (30) applicator tips, (50) applicator brushes, (1) mixing well	

#### ACCESSORIES & REFILLS

S390	Dispensing Tip (Pre-Bent/Yellow) Refill (30 pcs)
S391	Fine (1.5mm) Micro Brush Applicator Refill (100 pcs)
S373	Plastic Mixing Well (2-pack)



For more information, visit [Parkell.com](http://Parkell.com) or contact your authorized Parkell distributor.

**parkell<sup>®</sup>**